

### Instructions for Use

1. Align front of lid (lid opening) to front of base (label).  
*If snaps do not fit with holes, check to make sure the lid and base are both facing forward.*
2. Press lid to base. Make sure all connections are secured.
3. Place syringe into opening. Pull tab to drop into container.
4. Replace container when fill level is reached. Insert tabs into slots for final closure.
5. Dispose of container according to hospital policy.

### For Use with Wall Bracket

#### Model 435 020

1. Gently slide container down into bracket until it clicks.
2. To remove container from bracket, insert the bracket key (Part # 440 020) into slots at bottom of unit and push container up.

#### Needle Key Feature

With container mounted in bracket, insert needle completely into slot and slide needle holder to right or left, engaging the needle hub into key. Rotate counter-clockwise until needle is unscrewed from holder. With blunt tip of syringe/holder, nudge the hub back to the center hole to release needle into container.

Note: The needle key feature must only be used when the container is secured in wall bracket.

